Fiche technique

CONFORT et SECURITE > Miroir

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GB ⚠ Do not install this equipment before reading the following instructions: **Fitting instructions.** Recommended height for the lower part of mirror: 1,05 m. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 3 fixing points have to be necessarily respected. **Care and maintenance.** The mirror's support functions correctly if it is fitted according to instructions. It accepts square mirrors from 400 x 400 mm to 700 x 700 mm, rectangular ones, diagonals from 500 to 1000 mm (length inferior or equal twice as big as the width), round ones from Ø 500 mm to 1000 mm, thickness from 4 to 6 mm. Simple adjusting system in position by notched disc. After fitting and before using, make sure that the fixing screws together with mirror's fixings are tightened securely. The handle has to be used only to orientate and tilt the mirror but in any case as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. When cleaning, hold the mirror during all the process. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this mirror's support. At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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« © PELLET ASC - Document non-contractuel »
Distributeurs : PELLET ASC et GODONNIER SAS