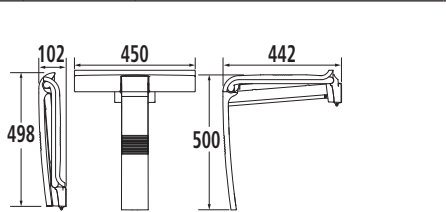
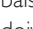
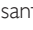
 Poids maximum utilisateur / Max. user weight / Maximengewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя.  Poids net / Net weight / Nettogewicht / Peso neto / Peso netto / Peso líquido / Nettogewicht / Вес нетто

									
Ce produit est conforme avec la directive 93/42/EEC des dispositifs médicaux. This product is conform with the directive 93/42/EEC for medical products. Dit product beantwoordt aan Richtlijn 93/42/EEC over medische apparatuur. Este producto es conforme con la Directiva 93/42/EEC relativa a los productos médicos. Questo prodotto è in conformità con la Direttiva 93/42/EEC relativa ai dispositivi medicali. Este produto está conforme à Directiva 93/42/EEC relativa aos dispositivos médicos Dieses Erzeugnis entspricht der Richtlinie 93/42/EEC über Medizinprodukte. Соответствует требованиям директивы 93/42 ЕС о медицинских изделиях.									
Référence G / Reference G / Referentie G / Referencia G / Codice G / Referência G / Referenz G / Код G :	<table><tr><td>135700</td><td>135716</td></tr><tr><td>135717</td><td>135719</td></tr><tr><td>135740</td><td>135840</td></tr><tr><td>135841</td><td></td></tr></table>	135700	135716	135717	135719	135740	135840	135841	
135700	135716								
135717	135719								
135740	135840								
135841									
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	<table><tr><td>047729</td><td>047730</td></tr><tr><td>047731</td><td>047732</td></tr><tr><td>047733</td><td>047734</td></tr></table>	047729	047730	047731	047732	047733	047734		
047729	047730								
047731	047732								
047733	047734								
Fonction Funcion Funcion Función Funzione Função Funktion Назначение	Permet de prendre une douche en position assise. Allows to take a shower in a seated position. Zo kan u zittend een douche nemen. Permite tomar una ducha en posición sentada. Permette di prendere una doccia in posizione seduta. Permite tomar um duche sentado. Ermöglicht es, im Sitzen zu duschen. Позволяет принимать душ в положении сидя.								
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание	 <table><tr><td></td><td>150 kg</td></tr><tr><td></td><td>2,600 kg</td></tr></table>		150 kg		2,600 kg				
	150 kg								
	2,600 kg								
Numéro notice / Instruction sheet no / Fichenummer / Número instrucciones / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	A13570016								
Date de création / Creation's date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	16/01/2013								
Révision n° / Revision no. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	07								
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	15/12/2017								

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GB  **Do not install this equipment before reading the following instructions:**

 **Fitting instructions.** Before drilling into the wall, physically hold the product (in the lowered position) against the wall so that the stand is PERPENDICULAR to the floor. Plugs have to be selected according to the type of wall. The 4 fixings points have to be necessarily respected.  **Care and maintenance.** **IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.  **Safety warnings.** The shower seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The shower seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. When the seat is in a raised position, **DO NOT USE** it as a grab bar. To avoid any pinching risks, do not

put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this shower seat.  At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant & Distributeur : GODONNIER SAS